# FeLV TEST INSTRUCTIONS FOR USE

FeLV TEST • INSTRUCCIONES DE USO

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FELV TEST • INSTRUKCE

## I.

serum/plasma • suero/plasma • sérum/plasma • Serum/Plasma • Sérum/plazma

## II.

whole blood • sangre completa • sanguine • Vollblut • Plná krev

SENSITIVITY AND SPECIFICITY  
INSECTA ET SPERMATOPHYTA  
study 2014  
Bacteria 2010

FeLV-Test	Enzym-linked Immunosorbent Assay	
	Sensitivity • Insecta	Specificity • Spermatochyta
	95 %	99 %

# FeLV TEST • INSTRUCTION FOR USE

The Henry Schein FeLV test is used to test for feline leukemia virus (FeLV) antigens in full blood, serum, and plasma.

### HANDLING THE SAMPLE

Do not keep the samples at room temperature for too long. Serum and plasma can be stored between 2-8°C for up to 3 days. For longer storage, the samples should be frozen at below -20°C. Venous full blood should be stored between 2-8°C, if the test is due to be carried out within 2 days from sampling. Do not freeze full blood samples. If samples are being sent somewhere, they should be packed in accordance with local regulations concerning aetiological agents. Bring the samples to room temperature before testing. Frozen samples must be fully thawed before testing and mixed well. Samples should not be refrozen and re-thawed.

### General instructions for the use of blood samples:

Separate the serum or plasma as rapidly as possible from the blood, to avoid haemolysis • Heparin blood and EDTA blood can be used to extract plasma • Only use clear rather than haemolysed samples • Testing should take place immediately after sampling

### I. TESTING IN THE CASE OF SERUM AND PLASMA

1. Take blood from the cat and extract the serum or plasma. Using the pipette, take up the sample material obtained. Add two drops of the sample material to the test field. Let the sample material soak in for a few seconds.
2. Then open the reagent vial and add two drops to the test field. The fluid begins to run up the test strip. If the fluid does not begin to run up after a few seconds, add a further drop to the reagent vial.

### II. TESTING FULL BLOOD

1. Take blood from the cat. Using the pipette, take up the sample material obtained. Add a drop of the sample material to the test field. Let the sample material soak in for a few seconds.
2. Then open the reagent vial and add two drops to the test field. The fluid begins to run up the test strip. If the fluid does not begin to run up after a few seconds, add a further drop to the reagent vial.

**ANALYSIS**  
After 10 minutes, the test can be analysed. One or two red lines appear in the reaction field.  
**POSITIVE TEST RESULT:**  
The test line and the control line become visible. **NB: even a faint test line must be interpreted as a positive test result.**

**NEGATIVE RESULT:**  
Only the control line becomes visible.  
**INVALID:**  
If no control line is shown, the test is invalid.  
**NB: the maximum time to reading is 20 minutes. All results read after 20 minutes are invalid.**

### DISPOSAL

All materials used must be disposed of carefully, because the material could be infectious. Test materials and used cassettes must be put in plastic bags and then disposed of.

### PLEASE NOTE

Only for animal use • Only for professional use • Use the cassette as soon as possible after opening • Please observe the required sample quantity • Do not apply any sample material to the reaction field • Do not touch the reaction field • Use only the fluid supplied • Do not use after the expiry date • Do not use the test cassette if the packaging is damaged • Observe analysis times indicated • Store and transport the test only at the temperatures indicated

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