**CPV/FPV Ag single test** 10 x #(2)800007  
Rev.: KI-40-00310-01

### CPV/FPV Ag single test

Rapid test

- PT Teste de CPV/FPV Ag
- CS CPV/FPV Ag Single Test
- PL Test CPV/FPV Ag
- RO CPV/FPV Ag Test unic
- SK CPV/FPV Ag Single Test
- DA CPV/FPV Ag Single Test
- NO CPV/FPV Ag enkelttest
- SV CPV/FPV Ag, enpack
- ZH CPV/FPV Ag 单项测试

### CPV/FPV Ag single test

Rapid test

- ES Test único de CPV/FPV Ag
- FR CPV/FPV Ag Single Test
- DE CPV/FPV Ag Einzeltest
- IT CPV/FPV Ag Single Test
- NL CPV/FPV Ag enkele test

For use in dogs and cats

Reorder # (2)800007

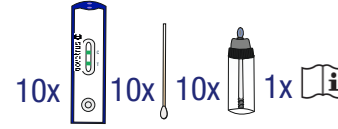
LOT 40003TTMMJJ

JJJJ-MM

30°C  
2°C

Distributed by:  
Covetrus BV  
Beversestraat 23  
5431 SL Cuijk (NL)  
cbproducts@covetrus.com

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Rev.: PO-40-003-01

CPV/FPV Ag single test  
single teststrip

LOT: 40003TTMMJJ Exp.: JJJJ-MM

Distributed by:  
Covetrus BV  
Beversestraat 23  
5431 SL Cuijk (NL)  
cbproducts@covetrus.com

# CPV/FPV Ag single test

Instructions for use

- ES Test único de CPV/FPV Ag · Instrucciones de uso
- FR CPV/FPV Ag Single Test · Mode d'emploi
- DE CPV/FPV Ag Einzeltest · Gebrauchsanweisung
- NL CPV/FPV Ag enkele test · Instructies voor 's use
- PT Teste de CPV/FPV Ag · Instruções per uso
- CS CPV/FPV Ag Single Test · Návod na použití

## EN CPV/FPV Ag single test

Instruction for use

The Covetrus CPV/FPV Ag single test is used to test rapidly and reliably for Parvovirus antigen in faeces samples.

### Handling the Sample

Fresh faeces requires no pre-treatment and can be examined directly. The samples can be cooled between 2-8°C for up to two days and stored at -20°C for several weeks. Before the test is carried out, the samples must be brought to room temperature. Frozen samples should be thoroughly mixed after thawing.

### Test procedure (Figure 1-4)

- Figure 1: Take a faecal sample using the cotton swab. Make sure that the upper end of the cotton swab is covered.
- Figure 2: Uncover the test tube and place the faecal cotton swab inside.
- Figure 3: Move the cotton swab back and forth in the liquid so that the faeces can solve. Then close and shake the test tube so that the sample and liquid mix well and dissolve.
- Figure 4: Break of the tip of the test tube by applying strong pressure. Open the test cassette packaging and place the cassette on a horizontal surface. Use gentle pressure to activate the cassette and position the sample in the reaction well. If the liquid does not run up on does not start after some seconds, add one more drop of the sample liquid into the sample well. If the liquid remains in the sample well and does not draw up, it helps to press slightly on the sample field with the back of the cotton swab.

### Analysis (Figure 5 and 6)

- After 10 minutes one or two red lines will appear in the reaction field, and the test can be analysed.
- Positive test result (Figure 5):**  
The test line and the control line become visible. Parvovirus is detected. NB: Even a faint test line must be interpreted as a positive test result. An excessive amount of sample can cause a brown line in the T-region (T), which must not be seen as a positive result. The test is invalid and should be repeated.
- Negative test result (Figure 6):**  
Only the control line becomes visible.
- Invalid:**  
If no control line is shown, the test is invalid. NB: All test results that are read later than 10 minutes are invalid.

### Disposal

All materials used must be disposed of carefully, because the material could be infectious. Test materials and used cassettes must be put in plastic bags and then disposed of.

### Please note

- For veterinary use only. • Only for professional use. • Use the cassette as soon as possible after opening. • Please follow the procedure for required sample quantity. • Do not apply any sample material to the reaction field. • Do not touch the reaction field. • Use only the fluid supplied. • To avoid cross-contamination, a separate collection vessel should be used for each sample. • Faecal samples may be infectious, so suitable precautions should be taken for handling and disposal-avoid skin contact and wear laboratory gloves and clothing. • Do not use after the expiry date. • Do not use the test if the packaging is damaged. • Observe analysis times indicated. • Store and transport the test only at the temperatures indicated.

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